

MIDAS REX[®] ELECTRIC BONE MILL SYSTEM



MIDAS REX[®] ELECTRIC BONE MILL SYSTEM

Features:

- Controlled, precise method to morselize bone
- Fast and efficient
- Maximizes processed bone yield
- Stand-Alone System
- Autoclaveable base
- Works with Autograft and/or Allograft

EASY TO SET UP AND USE*



Press bowl firmly onto Midas Rex Bone Mill base. Observe that both release buttons will be in the out position.



Place harvested bone specimens into bowl.



Secure cap onto bowl by rotating counterclockwise.



Run Bone Mill until desired particle size is achieved. Use spatula to remove bone particles.



Medtronic Powered Surgical Solutions
4620 North Beach Street
Fort Worth, TX 76137
USA
Internet: www.medtronic.com
Telephone: 817-788-6400
Toll-free: 1-800-643-2773
Fax: 817-788-6401

ORDERS:
Customer Service
1-800-433-7080
FAX: 817-788-6489

REPAIRS:
Repair Department
1-800-335-9557
817-788-6440
FAX: 817-788-6441

AUTHORIZED REPRESENTATIVE IN EU:
Medtronic B.V.
Earl Bakkenstraat 10
6422 PJ Heerlen
The Netherlands

Ordering Information:

BM110	Midas Rex [®] Bone Mill Base
BM120	Midas Rex Bone Mill Console
BM130	Midas Rex Bone Mill Instrument Tray
BM210	Midas Rex Dual Blade Disposable
EA600	North American 110 VAC Power Supply Cord

LIT250016EN

Federal law (USA) restricts this device for sale by or on the order of a physician. Patent number 6,824,087 B2.
©2006, Medtronic, Inc. All rights reserved. ®Registered marks of Medtronic, Inc.
*See Electric Bone Mill Instructions For Use for full operating procedure.

