

Maintenance & Sterilization

non-stick forceps:

Because there is no sticking, there is **no need to scrub SuperGliss® forceps** – thus **no scalpels, wire brushes, and no harsh or abrasive sponges are needed.**

Because SuperGliss® is softer than steel, the **tips should always be protected**, e.g. with protective tip caps or in a container.

Because SuperGliss® will oxidize, the tips may discolour and turn dark. The **oxidation layer is biocompatible and harmless**, for optical purposes it may be removed with a silver polishing cloth.

Call us if you have further questions: _____ / _____

SUPER
NON STICK **GLISS®**

