



# Espiner Tissue Retrieval System

## EcoSacs 170 to 240 - Instructions For Use


**1** Deploy the sac base first

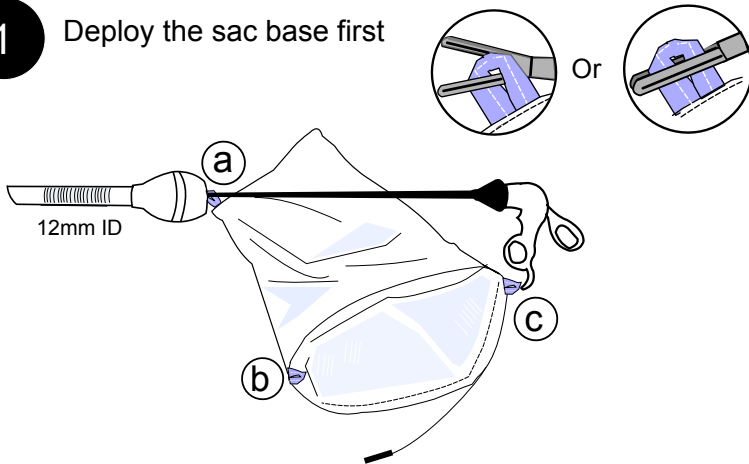


**2**

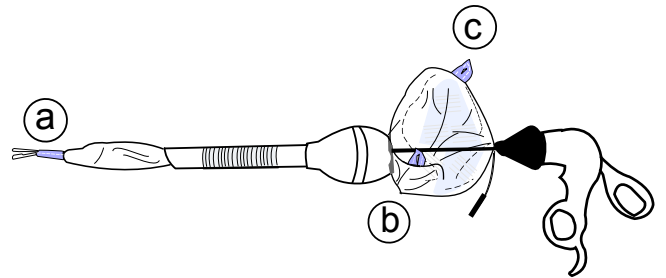


**3**

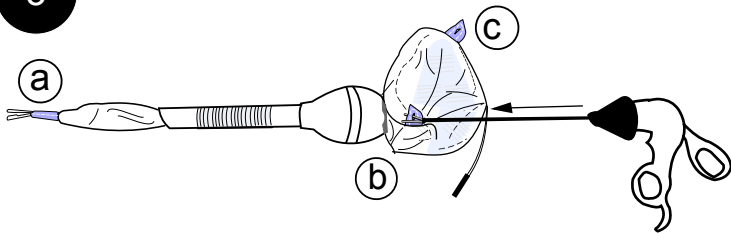


**4**

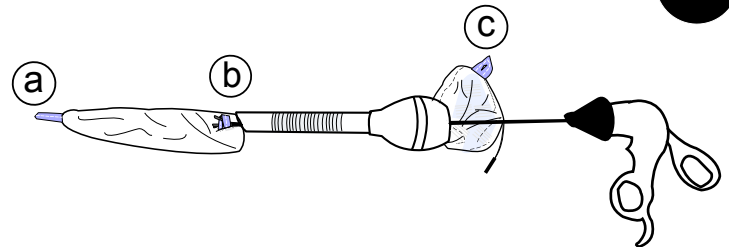


**5**

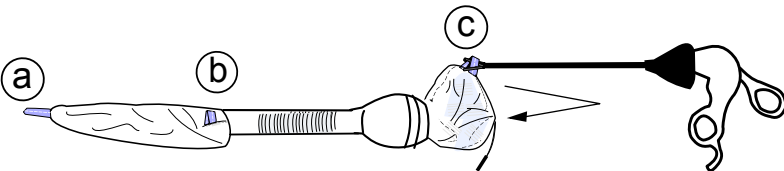


**6**

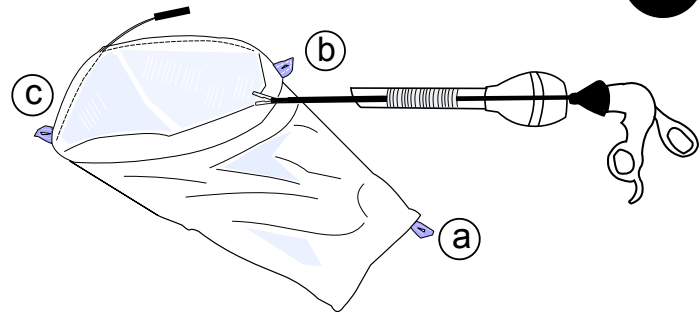

**7** Prepare to remove by closing mouth with drawstring

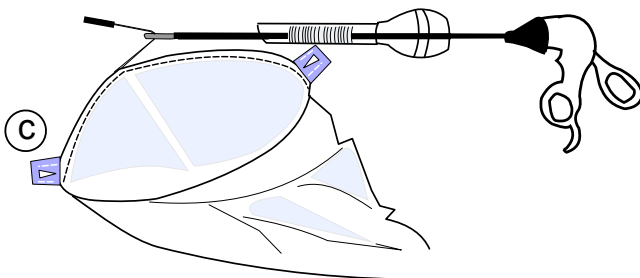

**8**

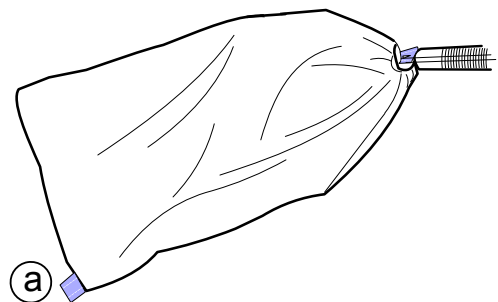

Use the drawstring by gripping the monofilament approximately 15mm from the heatshrink

Monofilament drawstring is designed for sac closure. Do not exceed the breaking strain of 25kg

Caution: Federal (USA) and European Union law restricts this device to sale, distribution and use by or on the order of a physician










0120 Consult instructions for use Recycle Single Use Only Not made with natural rubber latex Store in a cool place Do not use after expiry date shown on unit label Part no.E175/2

The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices