


FREEDOM60[®]

Syringe Infusion System

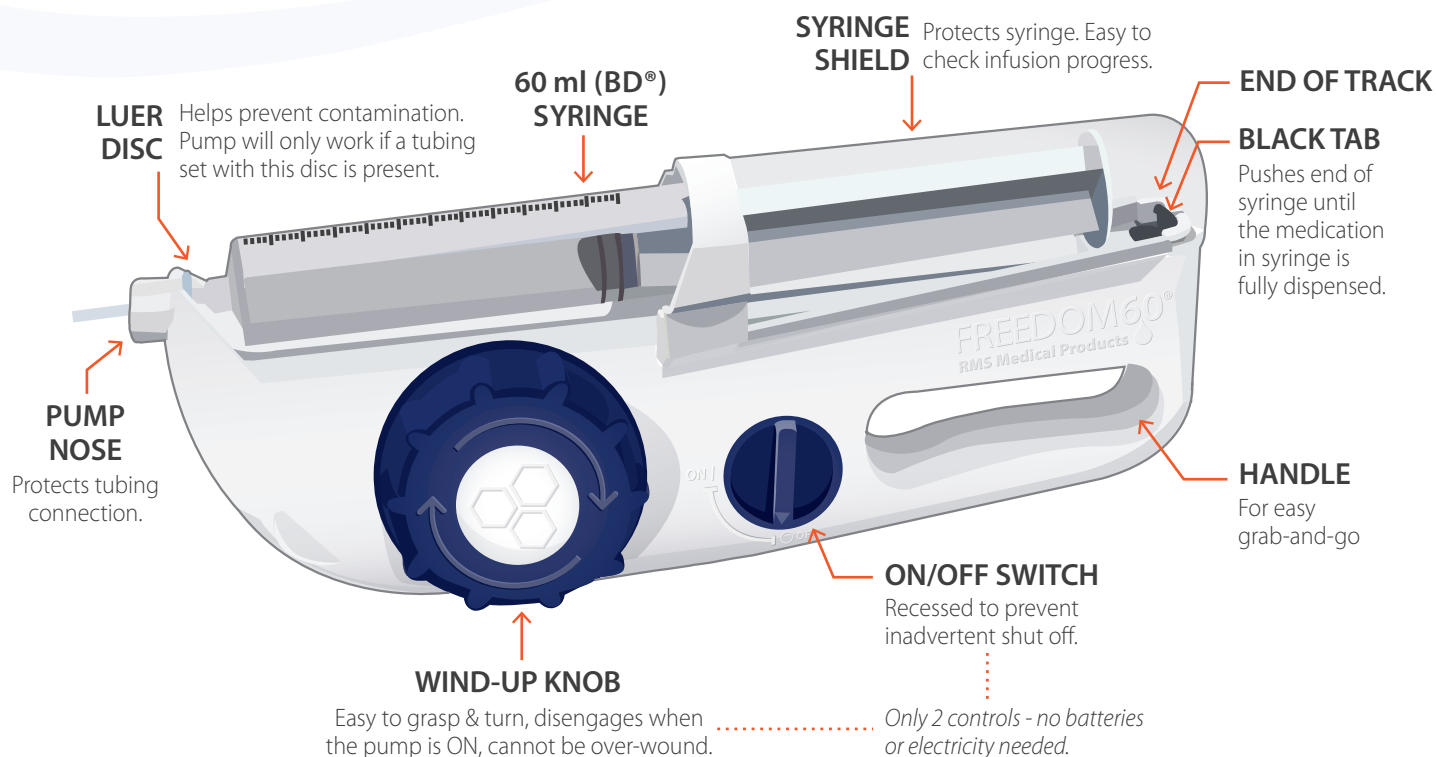


Indications For Use:

The FREEDOM60[®] Syringe Infusion System is indicated for the infusion of prescribed liquid medicines.

Infusions made easy as 1, 2, 3...

1 Wind

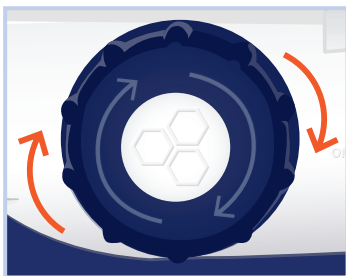


Wind knob until black tab is at end of its track.

2 Load



Load prepared syringe.

3 Go

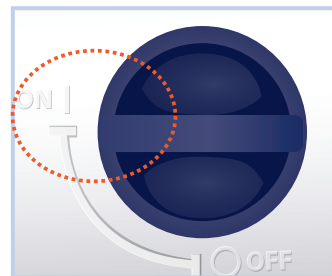

Switch ON and go!



rmsmedicalproducts.com

845-469-2042

800-624-9600

The FREEDOM60[®] and FreedomEdge[®] Syringe Infusion Systems, RMS Precision Flow Rate Tubing[™], and HigH-Flo Subcutaneous Safety Needle Sets[™] are registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2017 RMS Medical Products; All Rights Reserved. F60_FL_TechSheet_03-1126vA

FREEDOM60[®]

Syringe Infusion System

Easy to Use:

- Only two operating knobs:
 - *Large, ergonomic wind knob is easy to grasp and turn*
 - *Recessed ON/OFF switch*
- Just 3 steps – wind, load and go
- Completely portable:
 - *No batteries or electric needed*
 - *Travel pouch included for easy transport*
- No alarms or product maintenance

Integrated Safety:

- Maintains safe pressure of 13.5 psi (0.93 bar)
- Only RMS Freedom systems have DynEQ™ (Dynamic Equilibrium), which:
 - *Automatically responds to increasing pressure (caused by tissue saturation) by slowing down the flow rate during delivery*
- Holds full pressure after infusion is complete to prevent blood return or drug backflow
- Eliminates:
 - *Concerns of a bolus*
 - *Overflow*
 - *Overdose or runaway infusion*

Quality, Rugged Design:

- Made in the USA with highest level of quality control for defect-free product
- Robust design made with molded ABS plastic housing
- No sensitive electric parts
- Reliable system, consistent infusions

Versatile:

- Can be used for various administrations
- RMS Precision tubing sets from F0.5-F2400

Ordering Information

Product Description:

FREEDOM60[®] Syringe Infusion Pump with Travel Pouch

Replacement Travel Pouch - black

Zebra Print Travel Pouch - pattern

Item #:

F10050

F10090

F10080



Technical Specs

Pump:

Weight: 14 oz (0.4 kg)
Length: 12" (304 mm)
Width: 4.5" (114 mm)
Height: 1.6" (41 mm)

System:

Accuracy: +/- 7%
Residual volume: <0.4 ml
Max. operating pressure: 15 psi (1.03 bar)

Syringe:

Reservoir volume: 60 ml (max)
60 ml BD[®] or Monoject[®] syringes

Other Products:



FreedomEdge[®] Syringe Infusion System

for smaller infusion doses, 20 ml and 30 ml syringes



High-Flo Subcutaneous Safety Needle Sets™

available in 24G and 26G, 4 mm - 14 mm lengths and 1-6 needle leg sets



RMS Precision Flow Rate Tubing™

for flow rates from 0.5 to 2400 ml/hr



24" Extension Tubing Set (FEXT)

for those who need a longer length



rmsmedicalproducts.com

845-469-2042

800-624-9600

The FREEDOM60[®] and FreedomEdge[®] Syringe Infusion Systems, RMS Precision Flow Rate Tubing™, and High-Flo Subcutaneous Safety Needle Sets™ are registered trademarks of RMS Medical Products and are compliant with Medical Device Directive 93/42/EEC. RMS Medical Products is ISO 13485 certified. © 2017 RMS Medical Products; All Rights Reserved. F60_FL_TechSheet_03-1126vA