


Indications For Use:

RMS HighFlo Subcutaneous Safety Needle Sets™ are intended for the delivery of medication to the subcutaneous tissue.


How to use HighFlo needle sets:

1 Remove Cap

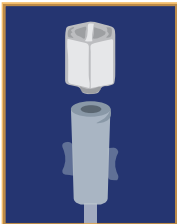


Remove the end cap from the female luer and connect the luer to the source of infusion.

2 Remove Closure

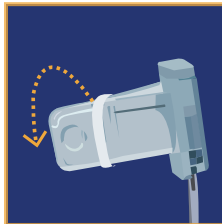

Remove the wings from the closure band and separate each wing.

3 Bend Back Wings

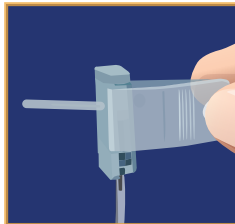


Bring the wings together and hold them between two fingers so that the needle guard is exposed.

4 Insert Needle




Remove the needle from the guard and insert into the subcutaneous tissue; cover with adhesive dressing.

5 Remove Needle

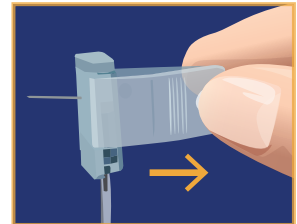

When infusion is complete, remove adhesive and pull needle out by bringing the wings together. Snap closed to enclose needle and discard.

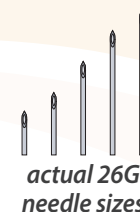

Safety Sets

- Safety closure protects against accidental needle stick injuries
- Emergency slide clamp included with every set
- Genuine 3M Tegaderm™ dressing to keep needles in place during infusion
- Made in the USA with highest level of quality control for defect-free product

Enhanced Design

- Backcut needle tip
 - *Minimizes tissue damage, scarring and local site reactions*
 - *Promotes faster healing*
- Custom luer provides even flow to every site
- Flexible flying hinge butterfly wings fit to patients body and keep needle in place without tenting

Versatile

- Needle lengths available in 4, 6, 9, 12, and 14 mm
- 24 and 26 gauge options
- Y-Connector combines sets up to 8 sites or used to infuse with 2 different needle lengths
- Sets available in single, double, triple, quad, penta and hexa configurations

Ordering Information

26 Gauge Needle Sets

1-Needle Sets box of 20



Length	Item #
4 mm	RMS12604
6 mm	RMS12606
9 mm	RMS12609
12 mm	RMS12612
14 mm	RMS12614

2-Needle Sets box of 10



Length	Item #
4 mm	RMS22604
6 mm	RMS22606
9 mm	RMS22609
12 mm	RMS22612
14 mm	RMS22614

3-Needle Sets box of 10



Length	Item #
4 mm	RMS32604
6 mm	RMS32606
9 mm	RMS32609
12 mm	RMS32612
14 mm	RMS32614

4-Needle Sets box of 10



Length	Item #
4 mm	RMS42604
6 mm	RMS42606
9 mm	RMS42609
12 mm	RMS42612
14 mm	RMS42614

5-Needle Sets box of 10



Length	Item #
4 mm	RMS52604
6 mm	RMS52606
9 mm	RMS52609
12 mm	RMS52612
14 mm	RMS52614

6-Needle Sets box of 10



Length	Item #
4 mm	RMS62604
6 mm	RMS62606
9 mm	RMS62609
12 mm	RMS62612
14 mm	RMS62614

24 Gauge Needle Sets

1-Needle Sets box of 20



Length	Item #
6 mm	RMS12406
9 mm	RMS12409
12 mm	RMS12412
14 mm	RMS12414

2-Needle Sets box of 10



Length	Item #
6 mm	RMS22406
9 mm	RMS22409
12 mm	RMS22412
14 mm	RMS22414

3-Needle Sets box of 10



Length	Item #
6 mm	RMS32406
9 mm	RMS32409
12 mm	RMS32412
14 mm	RMS32414

4-Needle Sets box of 10



Length	Item #
6 mm	RMS42406
9 mm	RMS42409
12 mm	RMS42412



Low Residual Volume Y-Connector - box of 25


Item # LRVY Combine sets up to 8 sites or infuse with 2 different needle lengths.

Residual Volumes

26 Gauge	1 needle	2 needle	3 needle	4 needle	5 needle	6 needle
Res. Vol.	.1 ml	.2 ml	.3 ml	.4 ml	.5 ml	.6 ml

24 Gauge	1 needle	2 needle	3 needle	4 needle
Res. Vol.	.4 ml	.7 ml	1.1 ml	1.4 ml

Other Products:

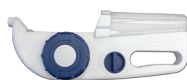

FREEDOM60® Syringe Infusion System
for larger infusion doses, 60 ml syringe



RMS Precision Flow Rate Tubing™
for flow rates from 0.5 to 2400 ml/hr



FreedomEdge® Syringe Infusion System
for smaller infusion doses, 20 ml and 30 ml syringes



24" Extension Tubing Set (FEXT)
for those who need a longer length