



LiNA Anti-Fog Solutions

Provides fog-free camera optics during endoscopic surgery

LiNA Anti-Fog Solutions

LiNA Medical's range of anti-fog solutions consists of LiNA Elfred, LiNA Clear Sight and LiNA Clear Sight Wipe.

LiNA Elfred

LiNA Elfred consists of a small sponge in an aluminium foil container with adhesive on the back. The sponge is pre-filled with 6ml anti-fog solution. LiNA Elfred offers a high degree of convenience as it is simple and ready for use:

- 1) Remove the adhesive strip under the container and attach it on the patient drape or instrument table.
- 2) Remove the lid of the container.
- 3) Press the scope onto the sponge and twist it to apply a membrane of the anti-fog solution.
- 4) If required, the sponge can be taken out of the container to clean the sides of the scope.

LiNA Elfred is available with or without alcohol. The alcohol version has a longer shelf life.

LiNA Clear Sight

Clear Sight consists of a 6ml bottle and a sponge with adhesive on the back. Clear Sight is easy to use:

- 1) Attach the sponge on the patient drape or instrument table.
- 2) Open the bottle and pour solution onto the sponge.
- 3) Press the scope onto the sponge and turn it to apply a membrane of the anti-fog solution.

The LiNA Clear Sight is available with or without alcohol. The alcohol version has a longer shelf life.

LiNA Clear Sight Wipe

The Clear Sight Wipe is pre-filled with anti-fog solution and individually packed in a peel pouch.



LiNA Clear Sight Wipe



LiNA Elfred



LiNA Clear Sight

Ordering Information:

LiNA Anti-fog solutions **0459**

REF.	Product	Packing
CS-20	LiNA Elfred - standard, single use.	Box of 30, sterile
CS-20A	LiNA Elfred - with 4% alcohol, single use.	Box of 30, sterile
CS-30	LiNA Clear Sight - standard, single use.	Box of 20, sterile
CS-30A	LiNA Clear Sight - with 4% alcohol, single use.	Box of 20, sterile
CS-40	LiNA Clear Sight Wipe - standard, single use.	Box of 30, sterile

LiNA distributor in your country:

