A.4113

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*Disposable medical devices
for rectosigmoidoscopy.*

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A complete range of single-use, diagnostic and paediatric rectoscopes with multi-use or single-use grip equipped with a magnifying glass and an attachment for air insufflator.

Easy to use, handy, perfectly smooth, atraumatic.

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A.4525VLNS

Kit Rettoscopi a LED.

Kit Rectoscopes with built-in LED light.

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Sapi Med disposable rectoscopes with built-in LED light can be used by the specialist either in out-patients' clinic and in operating room (the sterile version). Tubes are smooth and made in plastic material with graduation. Each tube has a grip with magnifying lens, connector for insufflator and built-in LED light source. By removing the transparent tab the LED powers on. Guaranteed same light intensity for two hours usage. SAPI MED grips are compatible, through adapter, to systems of image acquisition (cameras) **AL-READY AVAILABLE IN HOSPITALS** (for example operating theatres laparoscopic column).

ALL RECTOSCOPES WITH BUILT-IN LED LIGHT ARE AVAILABLE IN THE STERILE VERSION.

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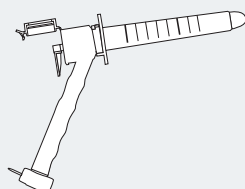
TECHNICAL FEATURES

- Flash free
- Guaranteed atraumatic
- Not rechargeable built-in LED light source
- External cm marking on tubes
- Separable grip and LED lighting for safe disposal
- They are made of plastic material conforms to the legislation 10993-1 regarding the biocompatibility
- Disposable grip
- Latex free
- Non sterile / Sterile

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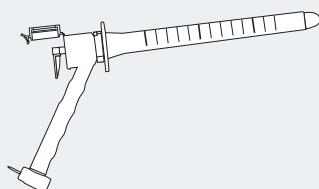
Transparent tube
Length: 130mm
Internal diameter: 17mm
External diameter: 19,6mm

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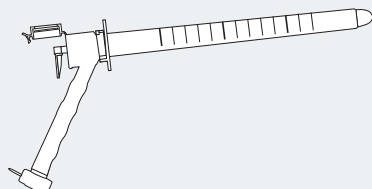
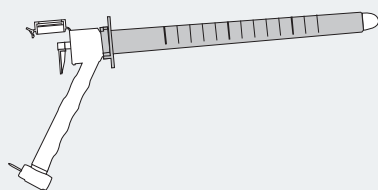
Transparent tube
Length: 250mm
Internal diameter: 17mm
External diameter: 19,6mm

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Green tube
Length: 250mm
Internal diameter: 17mm
External diameter: 19,6mm

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Kit Rectoscopes with built-in LED light with insufflator.

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The **disposable rectoscopes of Sapi Med** with built-in LED light and air insufflator can be used by the specialist either in out-patients' clinic and in operating room (the sterile version). Tubes are smooth and made in plastic material with graduation. Each tube has a grip with magnifying lens, connector for insufflator, air insufflator and built-in LED light source. Removing the transparent tab the LED powers on. Guaranteed same light intensity for two hours usage. Compatible, through adapter, to systems of image acquisition (cameras) **ALREADY AVAILABLE IN HOSPITALS** (for example operating theatres laparoscopic column).

ALL RECTOSCOPES WITH BUILT-IN LED LIGHT ARE AVAILABLE IN THE STERILE VERSION.

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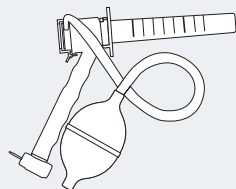
TECHNICAL FEATURES

- *Flash free*
- *Guaranteed atraumatic*
- *Not rechargeable built-in LED light source*
- *External cm marking on tubes*
- *Separable grip and LED lighting for safe disposal*
- *They are made of plastic material conforms to the legislation 10993-1 regarding the biocompatibility*
- *Disposable grip*
- *Disposable insufflator*
- *Latex free*
- *Non sterile / Sterile*

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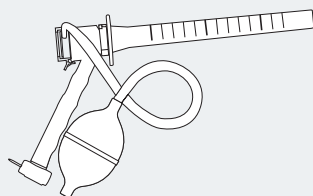
Quantity per box:
 20 tubes + 20 grips + 20 insufflators

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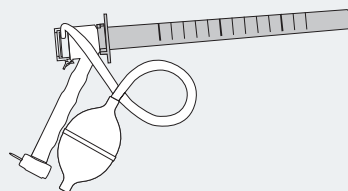
AVAILABLE IN NON STERILE VERSIONS AS
WELL.

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***Multi-Purpose Rectoscopes for
endorectal ecography and biopsies.***

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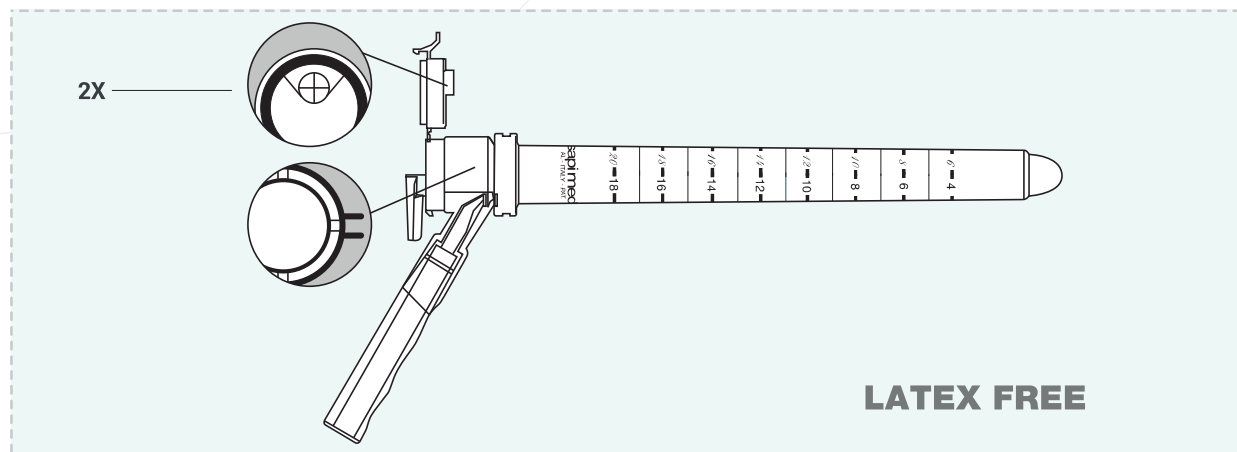
*Sapi Med announces new scopes design are to be added to their extensive coloproctology range. The new disposable scopes **MPR** and **MPR Short** feature a double-graduated silkscreen on their external surface to enable it to be used for controlled exploration of the rectum, and as an effective means for more thorough diagnostic examinations with dedicated instruments.*

Below the lens lies a membrane, with a cross marking a weakened point. This makes the handling of forceps for mucosa biopsy easier, whilst keeping the rectum walls stretched. Rectoscopes diameter and length have been designed to allow the insertion of a suitable probe for endorectal ecography for an accurate diagnosis of the rectal ampulla.

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TECHNICAL FEATURES

- Flash free
- Guaranteed atraumatic
- Compatible with all Sapi Med light sources
- External cm markings on tubes
- They are made of plastic material in compliance with legislation 10993-1 regarding the biocompatibility
- Grip with lens 2X
- Latex free, single-use, single-patient
- Air insufflator

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INSTRUCTIONS FOR USE

1. Insert the black pen-shaped end of the optical fibre light cable into the hollow handle of the rectoscope: connect the other end to the Sapi Med's cold light generators to ensure optimal lighting of the examination area;
2. lubricate the MPRs and obturators using an inert gel (Lu J, Sapi Med);
3. gradually introduce the scope into patient's anus: depth can be checked reading the "roman" silk-screen;
4. remove the obturator and attach the grip with a clockwise twist;
5. close the grip lid with light thumb pressure applied to the black lens frame;
6. attach the bellows set/disk filter attachment to the small cylindrical tube found on the left part of the lid, and insufflate air into the rectum to stretch the mucosa;
7. surgical examination can then begin.

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Length: 190mm
Internal diameter: 18,5mm
External diameter: 20,9mm
Quantity per box: 12 pcs.
Sterilised by: ETO

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Only for REF A.4522 and A.4522I: the two small fins found on the right of polymer grip are designed to anchor the tap to the rectoscope probe and allow the two devices rotate together. The "slanted" silk-screen allows adjustment of the probe during introduction into the rectum.

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Also available in non-sterile version for the codes: REF A.4519NS, A.4519INS, A.4522NS and A.4522INS

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Multi-Purpose Rectoscopes with build-in LED light for endorectal ecography and biopsy

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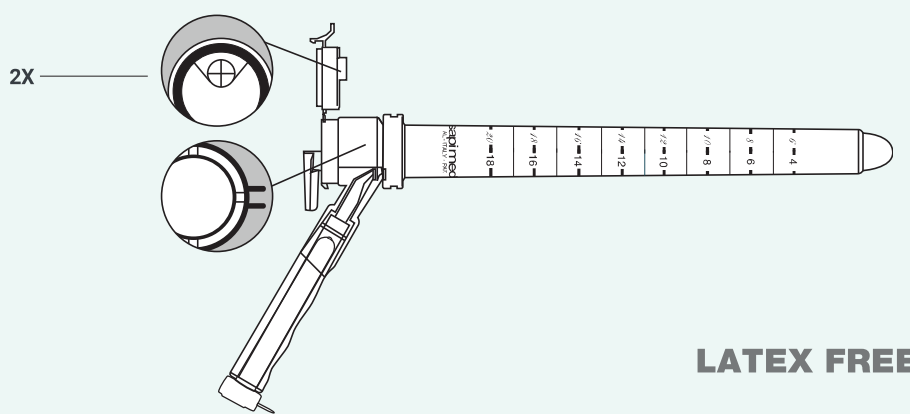
*Operative rectoscopes **MPR and MPR Short** were born to fulfill all proctologist's necessities. The double-graduated silkscreen on their external surface enable it to be used for controlled exploration of the rectum through ultrasound probe, but they are also two valid supports in performing further diagnostic exams which may require dedicated instruments. The devices feature a non-rechargeable LED self-lighting system. By removing the transparent tab the LED powers on. Guaranteed same light intensity for two hours usage.*

ALL MPR WITH BUILT-IN LED LIGHT ARE AVAILABLE IN THE STERILE VERSION.

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TECHNICAL FEATURES

- Flash free
- Guaranteed atraumatic
- External cm markings on tubes
- They are made of plastic material in compliance with legislation 10993-1 regarding the biocompatibility
- Grip with lens 2X
- Separable grip and LED lighting for safe disposal
- Latex free, single-use, single-patient
- Non Sterile / Sterile

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INSTRUCTIONS FOR USE

1. Lubricate the tube and obturator by using an inert gel (Lu J – Sapi Med);
2. Slowly and gently introduce the tube into the patient's anus up to the desired depth, which can be determined by observing the external graduation (normal one);
3. Remove the obturator and connect the grip to the tube with a clockwise rotation;
4. Close the grip lid by pressing with thumbs on the black lens frame;
5. Connect the insufflator to the small cylindrical tube found on the left side of the lid and insufflate air into the rectum in order to stretch the mucosa;
6. Rectal examination can then begin.

Below the lens lies a membrane, with a cross marking a weakened point. This makes the handling of forceps for mucosa biopsy easier, whilst keeping the rectum walls stretched. Rectoscopes diameter and length have been designed to allow the insertion of a suitable probe for endorectal ecography for an accurate diagnosis of the rectal ampulla. The two small fins found on the right of polymer grip are designed to anchor the tap to the rectoscope probe and allow the two devices to rotate together. The "slated" silk-screen allows adjustment of the probe during introduction into the rectum.

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Multi-Purpose Rectoscopes with build-in LED light and air insufflator for endorectal ecography and biopsy

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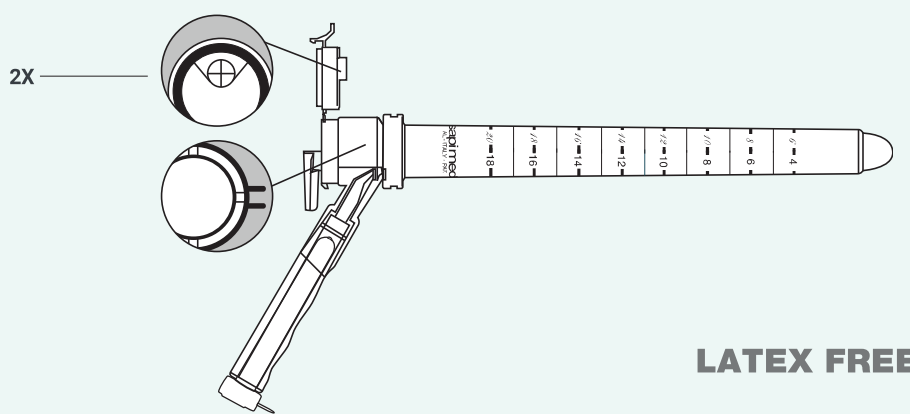
*Operative rectoscopes **MPR and MPR Short** were born to fulfill all proctologist's necessities. The double-graduated silkscreen on their external surface enable it to be used for controlled exploration of the rectum through ultrasound probe, but they are also two valid supports in performing further diagnostic exams which may require dedicated instruments. The devices feature a non-rechargeable LED self-lighting system. By removing the transparent tab the LED powers on. Guaranteed same light intensity for two hours usage.*

ALL MPR WITH BUILT-IN LED LIGHT AND INSUFFLATOR ARE AVAILABLE IN THE STERILE VERSION.

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TECHNICAL FEATURES

- Flash free
- Guaranteed atraumatic
- External cm markings on tubes
- They are made of plastic material in compliance with legislation 10993-1 regarding the biocompatibility
- Grip with lens 2X
- Separable grip and LED lighting for safe disposal
- Latex free, single-use, single-patient
- Air insufflator
- Non sterile / Sterile

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*Disposable insufflating bellow
for rectoscopes.*

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Peso: 60 g circa

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A.8200 non sterile

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Disposable insufflator bellow for air insufflations.

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Produttore: A.Hohlkorper - Germany

Distributore: Sapi Med s.p.a. - Italia

· *Insufflator double bellow for air insufflations*

· *Material: in soft PVC*

· *Insufflating capacity: 65 cm³*

· *Weight: 60 g*

Manufacturer: A. Hohlkorper - Germany

Distributor: Sapi Med s.p.a. - Italia