



Ultimate Perfection: intelligent solutions for each field of application

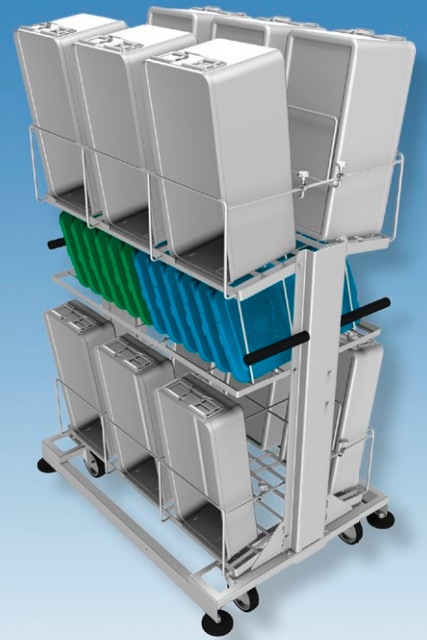

Comprehensive range of accessories for cleaning and disinfection
WD 750

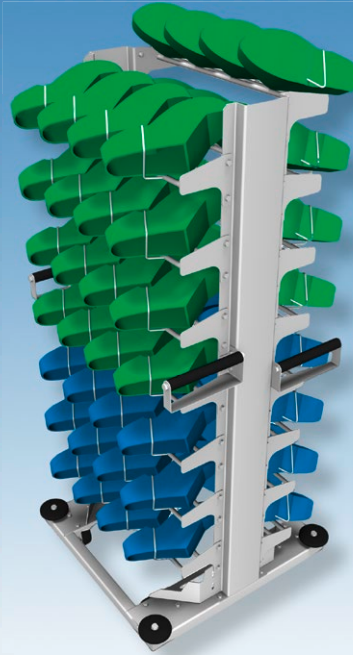

Intelligent accessories – clear added value to the workflow

Reprocessing today is expected to be fast, thorough and cost-effective. How is it accomplished? With systems and accessories that exactly meet those criteria.

Increasing performance, reducing costs

Often the focus is only on opting for an appropriate washer-disinfector. However, use of appropriate accessories is just as important. The basic requirements for a successful process are sufficient capacity and the performance of cleaning and drying. Nevertheless, with hundreds of manual steps in procedures, ergonomics and user protection are key issues in daily routine.

Expertise close to the customer

Numerous requirements – one contact partner: Belimed is the specialist for cleaning, disinfection and sterilization. We are one of the world's leading companies in this field and offer you over 40 years of experience in the development and manufacture of reprocessing systems. Our history of great expertise makes us a valuable partner for our customers. Your specific needs are in focus - we provide the tailor-made solution.

Cleverly designed features provide the best possible ergonomics for your daily work

Different items to be reprocessed and different applications mean different process requirements. Belimed collaborates closely with users in order to make life as easy as possible for them.



Easy and safe handling

Compact, robust racks make work easier and provide optimal space utilization. Cleverly devised features focus on the user. **Non heat-conducting handles** are located at an ergonomic working height – an extra benefit in terms of occupational safety and handling. Easy loading ensures optimal utilization of capacity and superior results in terms of cleaning and drying – cycle after cycle. At the same time you can load racks with frames on a variable basis and thus have a space-saving option available for combinations.

All Belimed racks are fitted with **deflector rollers** to avoid damaging furniture, buildings and machines. Due to the patented Belimed **process status display**, staff always have key process data within view, e.g. remaining run time, readiness for loading/unloading, or error messages – it is clearly visible even from a distance of 20 meters. Glass sliding doors and a glare-free **illuminated washing chamber** make it possible to visually monitor the washing process at all times.

The right solution for all fields of application


Whether you have a large medical center or a hospital with small numbers of beds – and irrespective of whether space is limited or not – we offer the right equipment for every demand.



Universal rack fitted with interchangeable frames

for loading with sterile items containers, operating room shoes, bowls, etc.

- For CSSDs with limited space
- No “parking areas” are available for loading racks
- For a wide variety of wash items
- Various configurations to easily handle different load types

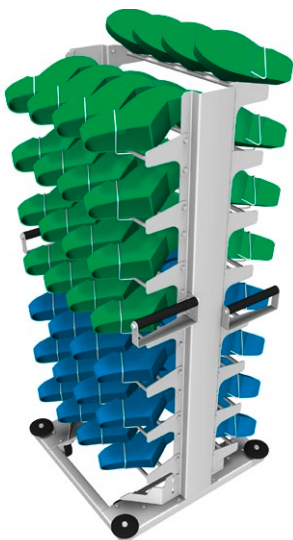


Special-purpose racks for any type of wash items

- High loading capacity
- Racks can be used as shelving (e.g. for storing operating room shoes)
- No extra set-up before loading
- High throughput

A precision engineered connection, the centerpiece of any system

The requirements for reprocessing increase steadily – a perfect-seal connection coupling with a patented sealing mechanism ensures excellent cleaning performance in every situation.

Only by Belimed

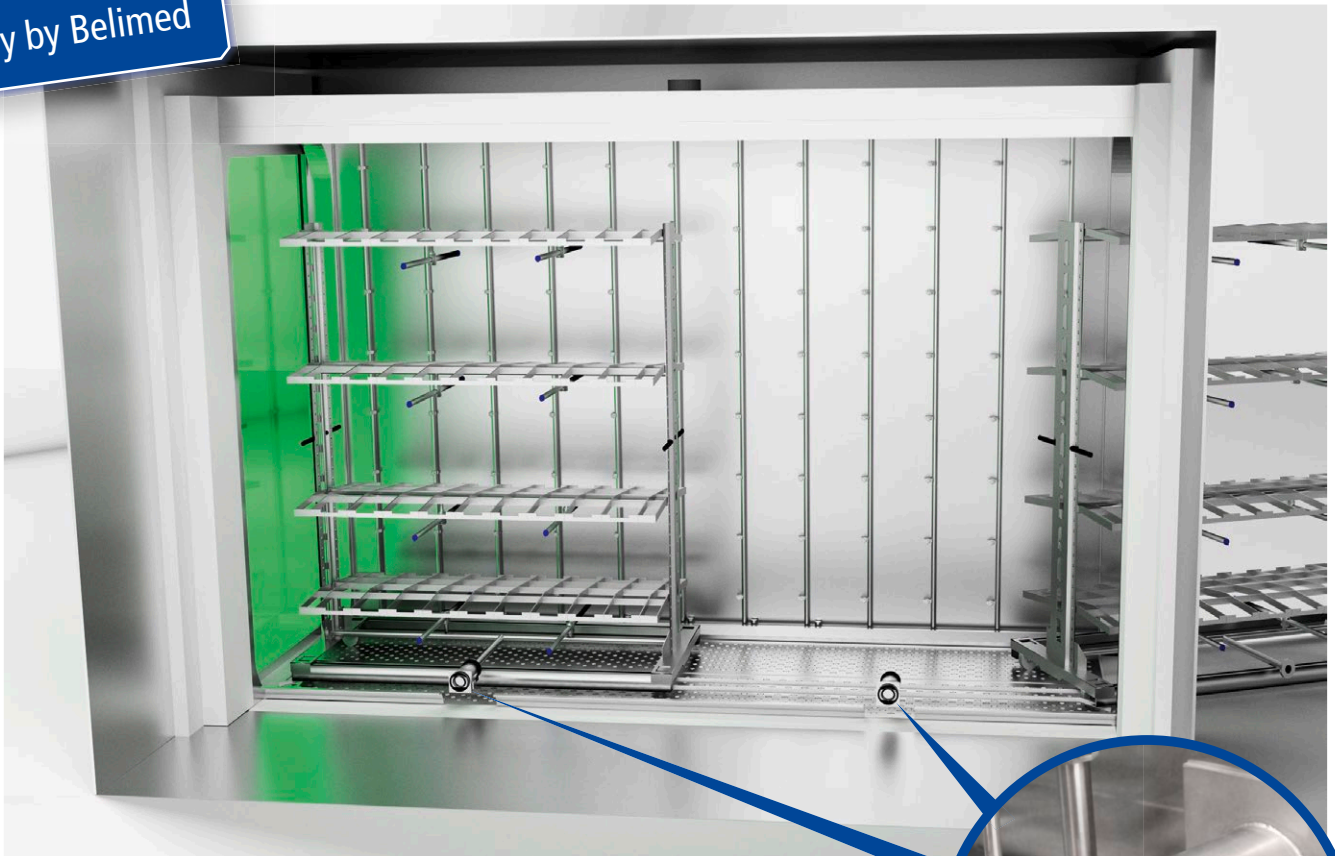


Since the items to be washed are sensitive and the standards of cleaning to be achieved are high, development work paid special attention to the cleaning system. Consequently, the cleaning and disinfection of wash items, e.g. surgical instruments, is accomplished through our precise connection coupling and the resulting uniform distribution of detergent solution and temperature.

The machine and rack combine to create a homogeneous integrated system. This is accomplished by a special docking device that connects the rack and cleaning system with a positive lock and thus ensures a constant-pressure, leak-free supply to the cleaning system.

On the one hand, you benefit from increased cleaning performance – on the other, the optimized process saves you valuable time and total capacity is increased.

A unique feature is the option of using two racks with a connection coupling. That means it is possible to both increase capacity and improve handling by using compact racks.

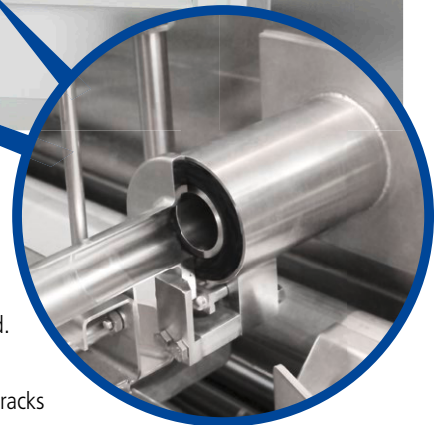


The Belimed universal rack – a system with plenty of options

Due to its enormous versatility and considerable flexibility the universal rack allows optimal utilization of capacity and cost-effectiveness. By simply changing the baskets a rack that has just been loaded with operating room shoes can be used for sterile items containers in the next cycle.



Universal rack for loading with operating room shoes – using individual frames

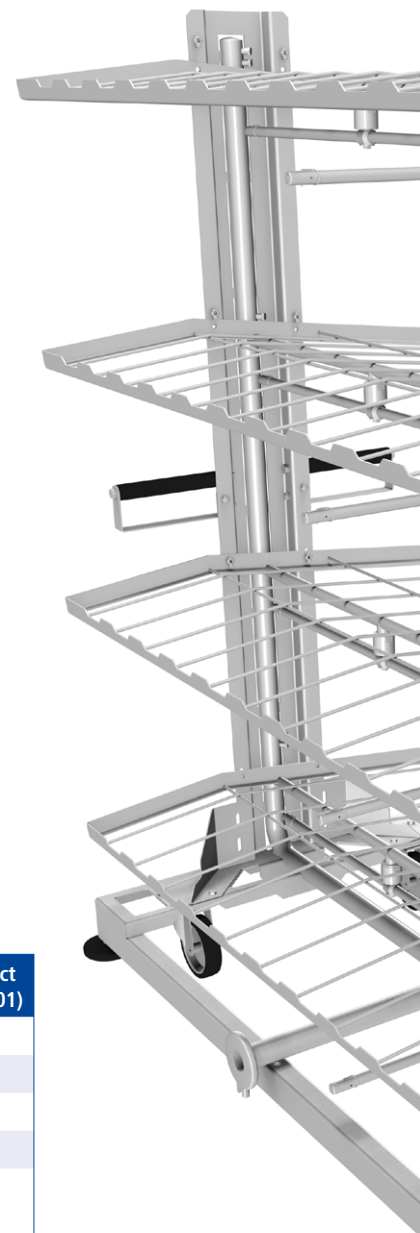


Loading capacity per universal rack	S rack (Art. No. 731 910)	M rack (Art. No. 730 763)	M rack compact (Art. No. 156 301)
Sterile container with lid (H × W × D = 300 × 600 × 300 mm)	20	27	13
Sterile items baskets for utensils (H × W × D = 340 × 570 × 185 mm)	16	18	12
Surgical instruments (DIN trays)	28	48	24
Operating room dishes, Ø 320 mm	32	36	24
OR shoes			
Up to size 42	240	270	180
Sizes 43 – 46	144	162	108



Universal rack for loading with sterile items containers



Universal rack for loading with operating room bowls – using individual frames

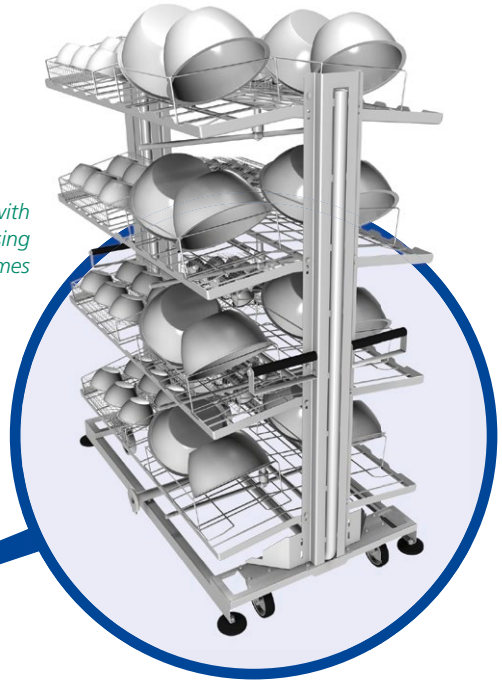

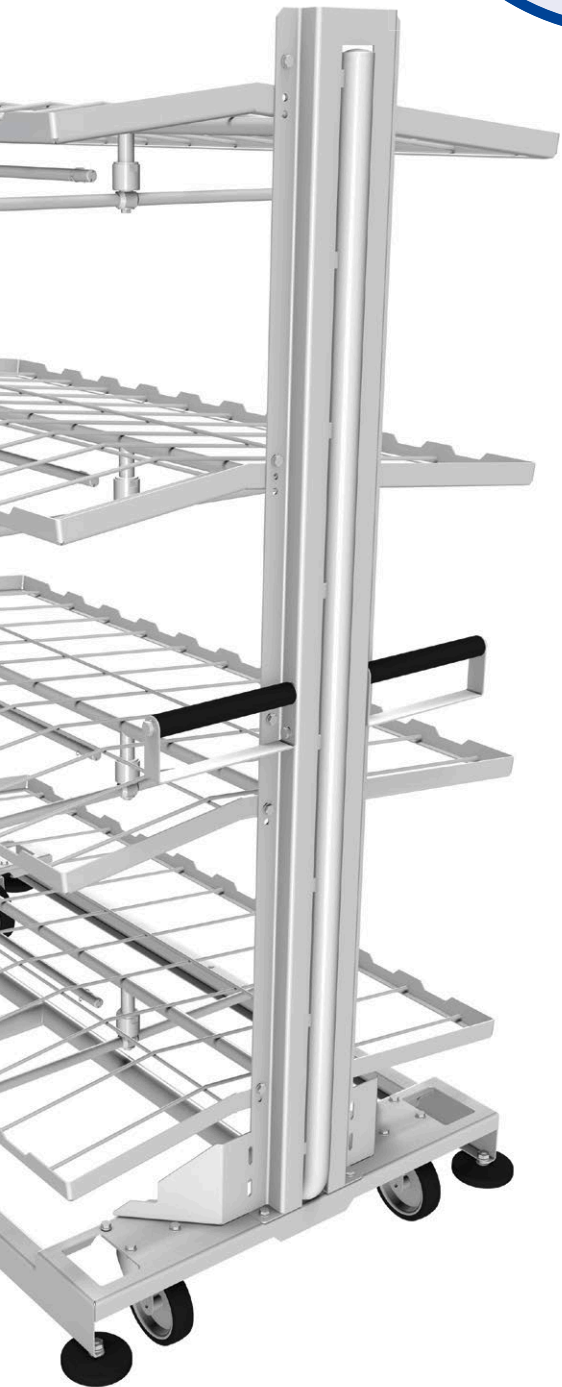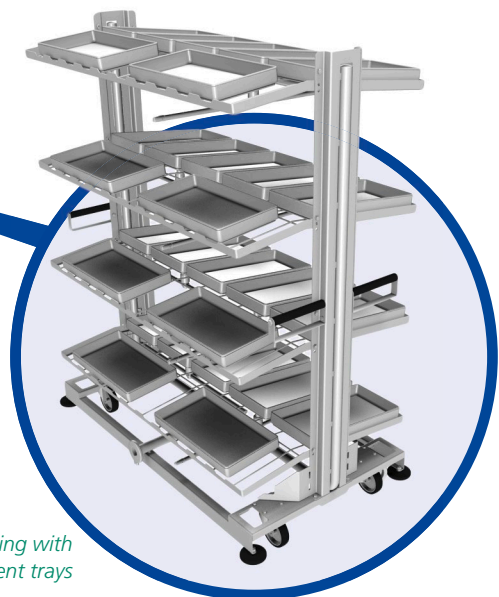


Universal rack for loading with instrument trays



Art. No. 731 910


Universal rack with internal feed
4-level basic rack for loading with different wash items






Dimensions H × W × D (mm)	1803 × 888 × 1488		
Capacity	See page 6		
Special features	<ul style="list-style-type: none"> • Universal use (containers, instruments, shoes, etc.) • 4 connections per level, e.g. for Luer Lock connectors 		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type			

Art. No. 730 782

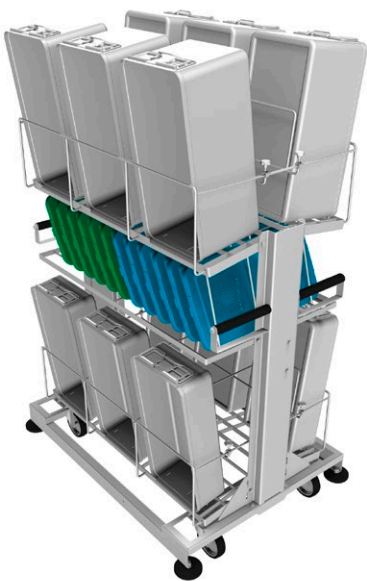Rack for sterile items containers
without internal feed




Dimensions H × W × D (mm)	1743 × 872 × 1150 (loaded)		
Capacity	12 sterile containers with lids (H × W × D = 300 × 600 × 300 mm)		
Special features	<ul style="list-style-type: none"> • Adjustable holder for high and low containers 		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type			

Art. No. 730 763

Universal rack with internal feed
4-level basic rack for WD 750M, for loading with different wash items






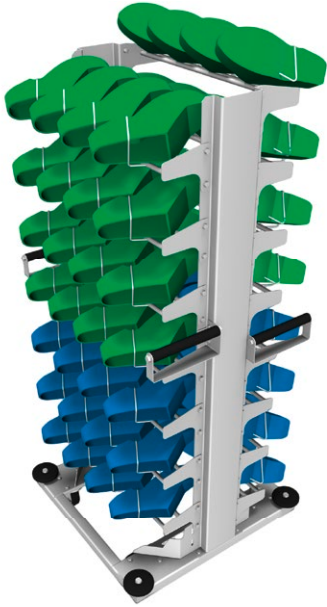



Dimensions H x W x D (mm)	1803 x 888 x 2128		
Capacity	See page 6		
Special features	<ul style="list-style-type: none"> • Universal use (containers, instruments, shoes, etc.) • 4 connections per level, e.g. for Luer Lock connectors 		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type	-		-




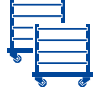
Art. No. 156 301

Compact universal rack with internal feed
4-level basic rack for WD 750M, for loading with different wash items



Dimensions H x W x D (mm)	1803 x 888 x 1140		
Capacity	See page 6		
Special features	<ul style="list-style-type: none"> • Compact design for easier handling • Flexible loading of 2 racks in one machine • 4 connections per level, e.g. for Luer Lock connectors 		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type			

Art. No. 732 390				Compact rack for operating room shoes without internal feed			
	Dimensions H × W × D (mm)	1730 × 750 × 750					
	Capacity	60 operating room shoes					
	Special features	<ul style="list-style-type: none"> • Compact design for easier handling • Can be used as a storage rack for stowing operating room shoes 					
	Design	Stainless steel Material no. 1.4301					
		WD 750S	WD 750M	WD 750L			
Capacity / machine type							

Art. No. 730 264				Rack for operating room shoes without internal feed			
	Dimensions H × W × D (mm)	1730 × 872 × 1488					
	Capacity	96 operating room shoes					
	Special features	–					
	Design	Stainless steel Material no. 1.4301					
		WD 750S	WD 750M	WD 750L			
Capacity / machine type							

Art. No. 731 090

Rack for stainless steel dishes

with internal feed



Dimensions H × W × D (mm)	1600 × 888 × 1488		
Capacity	32 stainless steel dishes (Ø 360 mm)		
Special features	–		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type			

Art. No. 731 088

Rack for aluminium boxes incl. lids

with internal feed




Dimensions H × W × D (mm)	1600 × 888 × 1488		
Capacity	14 aluminium boxes with lids H × W × D = 520 × 360 × 210 mm		
Special features	–		
Design	Stainless steel Material no. 1.4301		
	WD 750S	WD 750M	WD 750L
Capacity / machine type			

Art. No. 731 089		Rack for plastic boxes without lids with internal feed		
	Dimensions H × W × D (mm)	1600 × 888 × 1488		
	Capacity	16 plastic boxes without lids H × W × D = 600 × 400 × 300 mm		
	Special features	–		
	Design	Stainless steel Material no. 1.4301		
		WD 750S	WD 750M	WD 750L
Capacity / machine type				


Accessories

For loading the racks with very different interchangeable frames – the right solution for any requirement.

Art. No. 772 999 / 771 303		Basket open / lid		
	For loading with	Art. No. 772 999 <ul style="list-style-type: none"> • Open wire-mesh basket for bulky goods and utensils • H × W × D = 185 × 570 × 340 mm • Mesh size = 11 × 11 × 1.6 mm Art. No. 771 303 <ul style="list-style-type: none"> • Lid for wire-mesh basket art. No. 772 999 • W × D = 570 × 340 mm • Mesh size = 11 × 11 × 1.6 mm 		
		S-Rack (Art. No. 731 910)	M-Rack compact (Art. No. 156 301)	M-Rack (Art. No. 730 783)
	Capacity	16	12	18

Art. No. 730 043

Loading frame for sterile container lids

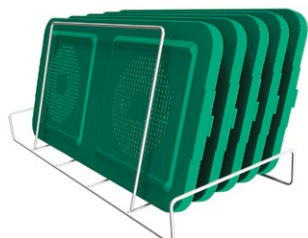

For loading with

- 5 sterile container lids

Features

H x W x D = 285 x 630 x 260 mm

Capacity

S-Rack
(Art. No. 731 910)

M-Rack compact
(Art. No. 156 301)

M-Rack
(Art. No. 730 783)

16

12

18

Art. No. 818 999

Basket for kidney dishes



For loading with

- 8 kidney dishes per basket

Features

H x W x D = 158 x 570 x 340 mm

Capacity

S-Rack
(Art. No. 731 910)

M-Rack compact
(Art. No. 156 301)

M-Rack
(Art. No. 730 783)

16

12

18

Art. No. 819 147

Basket for bowls, large



For loading with

- 2 bowls, Ø 320 mm, per basket

Features

H x W x D = 130 x 550 x 335 mm

Capacity

S-Rack
(Art. No. 731 910)


M-Rack compact
(Art. No. 156 301)


M-Rack
(Art. No. 730 783)


16

12

18

Art. No. 769 014		Basket for bowls, medium		
	For loading with	<ul style="list-style-type: none"> • 6 bowls, Ø 160 mm, per basket 		
	Features	H × W × D = 120 × 550 × 320 mm		
		S-Rack (Art. No. 731 910)	M-Rack compact (Art. No. 156 301)	M-Rack (Art. No. 730 783)
	Capacity	16	12	18

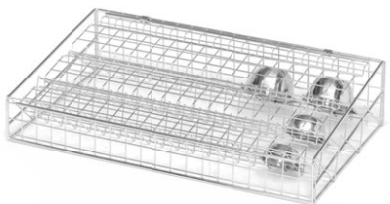Art. No. 771 312		Basket for bowls, small		
	For loading with	<ul style="list-style-type: none"> • 15 bowls, Ø 100 mm, per basket 		
	Features	H × W × D = 120 × 550 × 320 mm		
		S-Rack (Art. No. 731 910)	M-Rack compact (Art. No. 156 301)	M-Rack (Art. No. 730 783)
	Capacity	16	12	18

Art. No. 814 650		Basket for operating room shoes up to size 42		
	For loading with	<ul style="list-style-type: none"> • 15 operating room shoes per basket 		
	Features	H × W × D = 255 × 570 × 340 mm		
		S-Rack (Art. No. 731 910)	M-Rack compact (Art. No. 156 301)	M-Rack (Art. No. 730 783)
	Capacity	16	12	18

Art. No. 814 649

Basket for operating room shoes, sizes 43–46



For loading with

- 9 operating room shoes per basket

Features

H × W × D = 255 × 570 × 340 mm

Capacity

S-Rack
(Art. No. 731 910)
16M-Rack compact
(Art. No. 156 301)
12M-Rack
(Art. No. 730 783)
18

Art. No. 818 213

Basket for trays



For loading with

- 27 trays per basket
H × W × D = 27 × 235 × 150 mm
- 18 trays per basket
H × W × D = 51 × 250 × 200 mm

Features

H × W × D = 190 × 560 × 700 mm

Capacity

S-Rack
(Art. No. 731 910)
4M-Rack compact
(Art. No. 156 301)
4M-Rack
(Art. No. 730 783)
8

Art. No. 870 052

Universal basket for bowls, kidney dishes, and trays



For loading with

- Including 3 partitions (art. No. 870 053)

Features

H × W × D = 230 × 340 × 570 mm

Capacity

S-Rack
(Art. No. 731 910)
16M-Rack compact
(Art. No. 156 301)
12M-Rack
(Art. No. 730 783)
18

www.belimed.com

AUSTRIA

Belimed GmbH
Grüne Lagune 1
8350 Fehring
Tel. +43 3155 40 699 0
Fax +43 3155 40 699 10
info@belimed.at

BELGIUM

NV Belimed SA
Rue de Clairvaux 8
1348 Louvain-La-Neuve
Tel. +32 10 42 02 40
Fax +32 10 42 02 49
info@belimed.be

CHINA

Belimed Medical Equipment
(Shanghai) Co., Ltd
CaiLun Road 780,
5th floor, Room H
ZhangJiang Hi-Tech Park
201203 Pudong, Shanghai
Tel. +86 21 513 709 98
Fax +86 21 513 709 96
info@belimed.cn

FRANCE

Belimed SAS
Parc GVIO
330 Allée des Hêtres, Hall E
69760 Limonest
Tel. +33 4 37 41 63 03
Fax +33 4 37 41 63 04
info.pharma@belimed.fr

Branch Office

Belimed SAS
ZAC Saumaty Séon
19 rue Gaston Castel
13016 Marseille
Tel. +33 4 96 15 22 10
Fax +33 4 96 15 22 19
info.sud@belimed.fr

Branch Office

Belimed SAS
Parc Espale
1, av. Pierre Pflimlin
68390 Sausheim
Tel. +33 3 89 63 65 40
Fax +33 3 89 63 65 41
info@belimed.fr

GERMANY

Belimed Deutschland GmbH
Edisonstrasse 7a
84453 Mühldorf am Inn
Tel. +49 8631 9896 0
Fax +49 8631 9896 300
info@belimed.de

NETHERLANDS

Belimed B.V.
Vlambloem 65
3068 JG Rotterdam
Tel. +31 10 286 17 50
Fax +31 10 456 56 97
info@belimed.nl

SLOVENIA

Belimed d.o.o.
Taborska cesta 38 E
1290 Grosuplje
Tel. +386 1 7866 000
Fax +386 1 7866 011
info@belimed.si

SWITZERLAND

Belimed Sauter AG
Zelgstrasse 8
8583 Sulgen
Tel. 0848 55 88 11
Fax +41 71 644 86 01
contact@belimed.ch

UNITED KINGDOM

Belimed Limited
Unit 4 Newbuildings Place
Dragons Green Road
Shipleigh
West Sussex, RH13 8GQ
Tel. +44 1403 738 811
Fax +44 1403 730 830
info@belimed.co.uk

USA

Belimed, Inc.
2325 Charleston
Regional Parkway
Charleston, SC 29492
Tel. +1 843 216 7424
Fax +1 843 216 7707
info@belimed.us

OTHER COUNTRIES

Medical/Medical Lab
Belimed AG
Dorfstrasse 4
6275 Ballwil
SWITZERLAND
Tel. +41 41 449 78 88
Fax +41 41 449 78 89
info@belimed.ch

OTHER COUNTRIES

Pharma/Pharma Lab
Belimed Sauter AG
Zelgstrasse 8
8583 Sulgen
SWITZERLAND
Tel. +41 71 644 85 00
Fax +41 71 644 86 01
pharma@belimed.com