


Connect >>>

Continue taking **small, careful steps** to slide the blue insert into the SuperSeton, until it is all the way in.

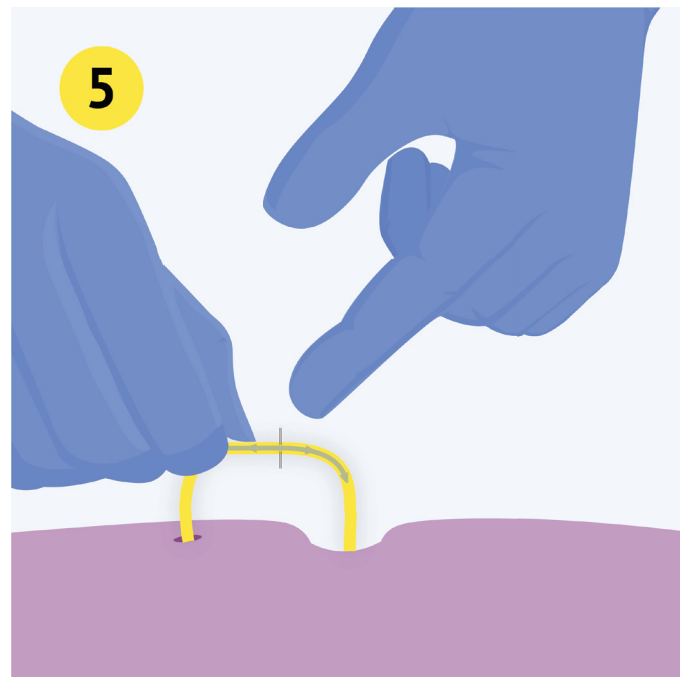


Check

Once the SuperSeton is closed, confirm that the insert is now **completely covered** by the tubing.



Ready!

That's it, you are all done. You can rest assured that the SuperSeton will stay in place and your patient has received the best possible care.

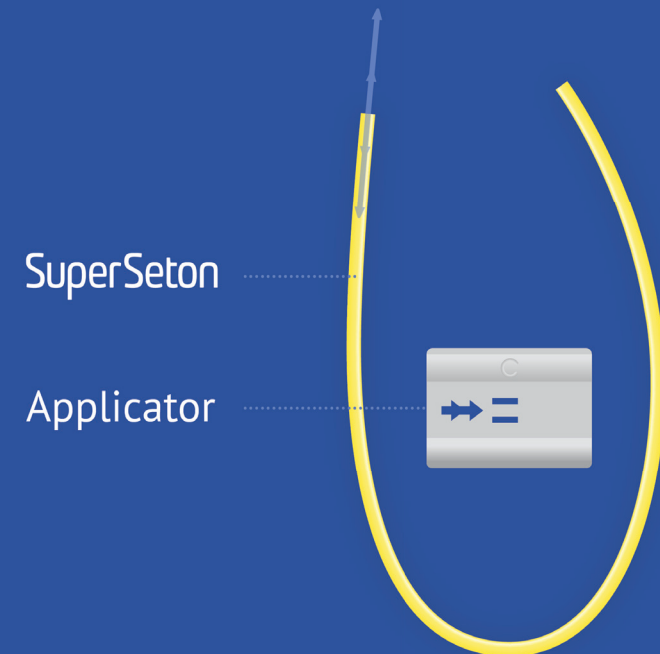

SuperSeton

Applicator


Included in every package, you'll find a SuperSeton and a disposable applicator.

Apply the SuperSeton in a few easy steps, as explained in this instruction leaflet.

SuperSeton

Smooth, simple, secure.


Instructions for use

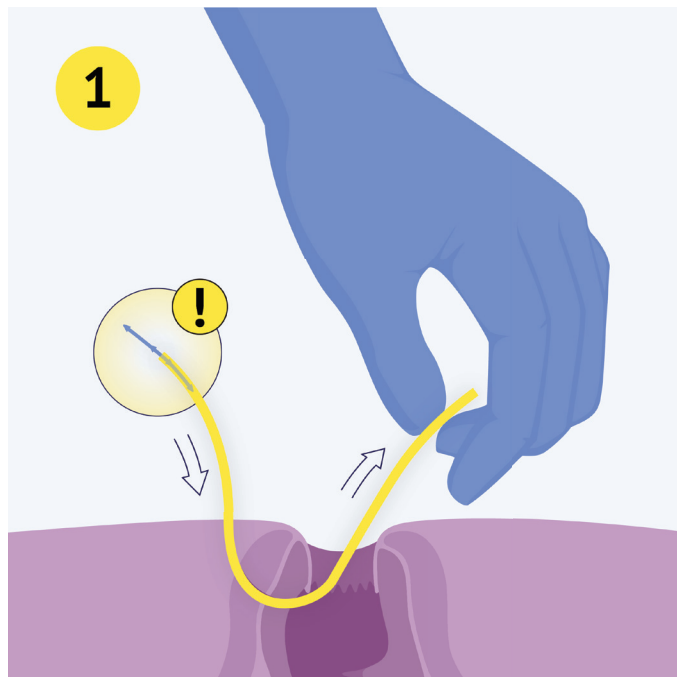

Insert


First, you insert the SuperSeton just like you would a regular seton.

Attention: do not lead with the blue insert!

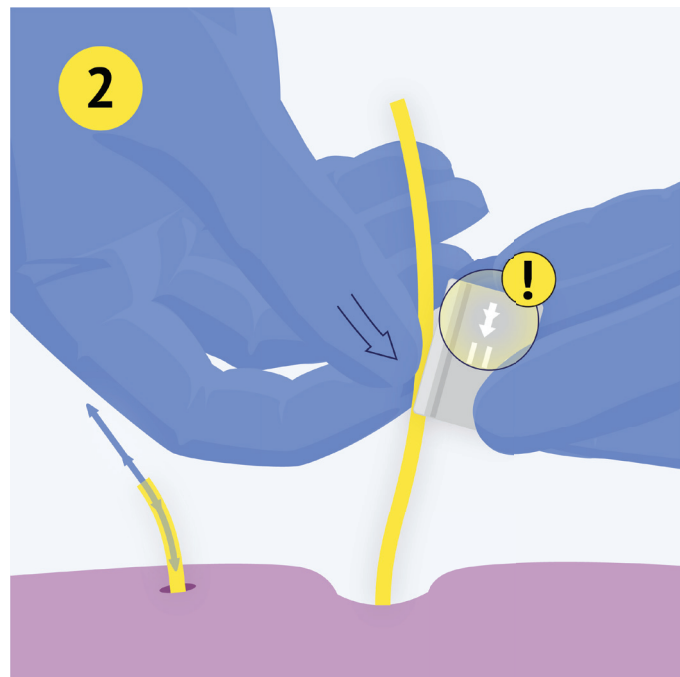


Hold

Place the open end of the SuperSeton (without the blue insert) into the applicator. The **arrow** symbol must point **towards the patient**.

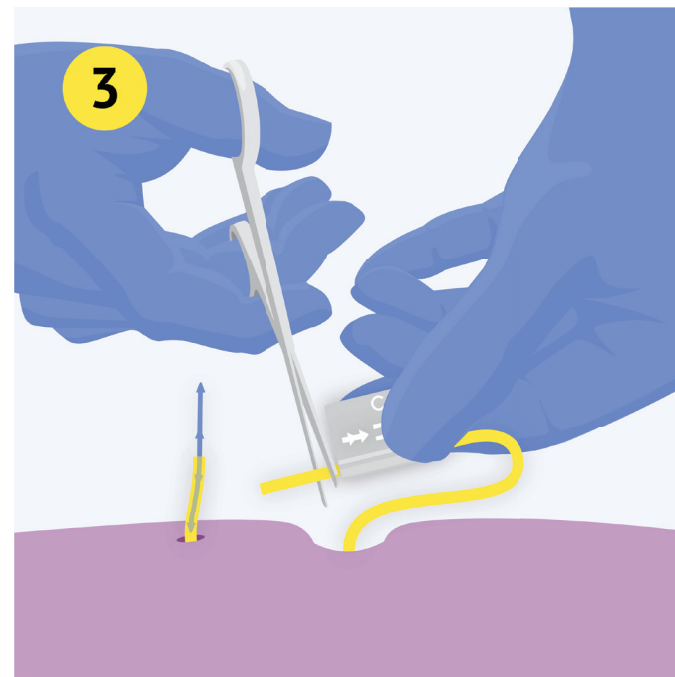


Trim

Now trim the seton to the appropriate size. The optimal length keeps the seton close to the body, without causing tension.

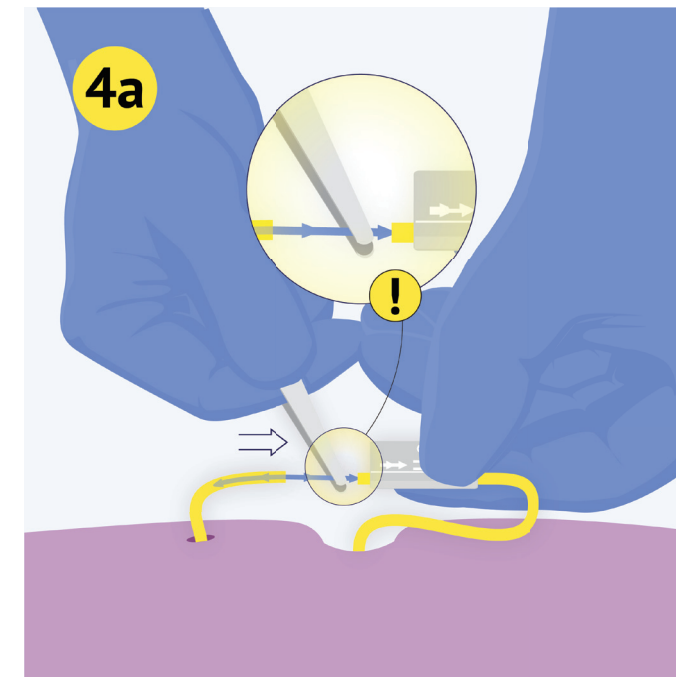

Connect >

Still holding the applicator, use broad tweezers to **grab the blue insert** at the end and carefully slide it into the open end of the SuperSeton.

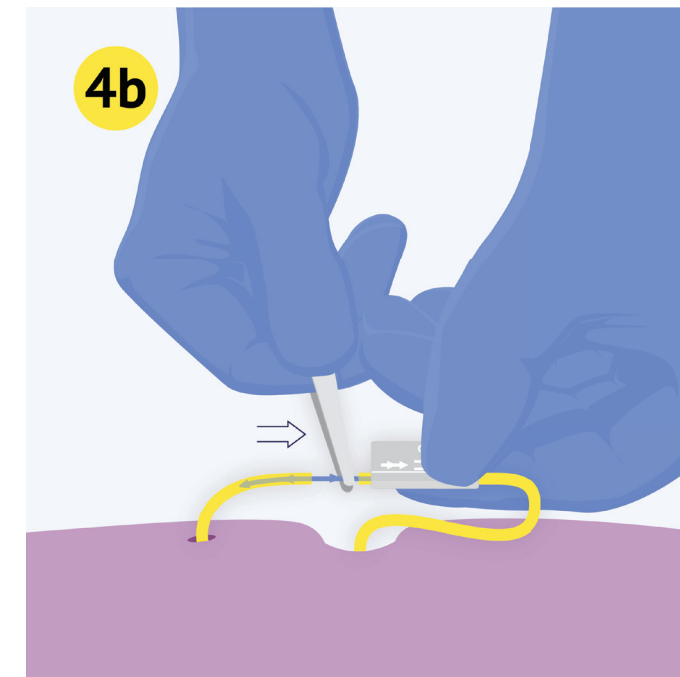

Connect >>

It will take a few steps to completely connect the SuperSeton. Always push gently and hold the insert close to the end, to **avoid bending**.