


SEROMA SET™

THE SOLUTION FOR
POST-OPERATIVE
SEROMA ASPIRATION

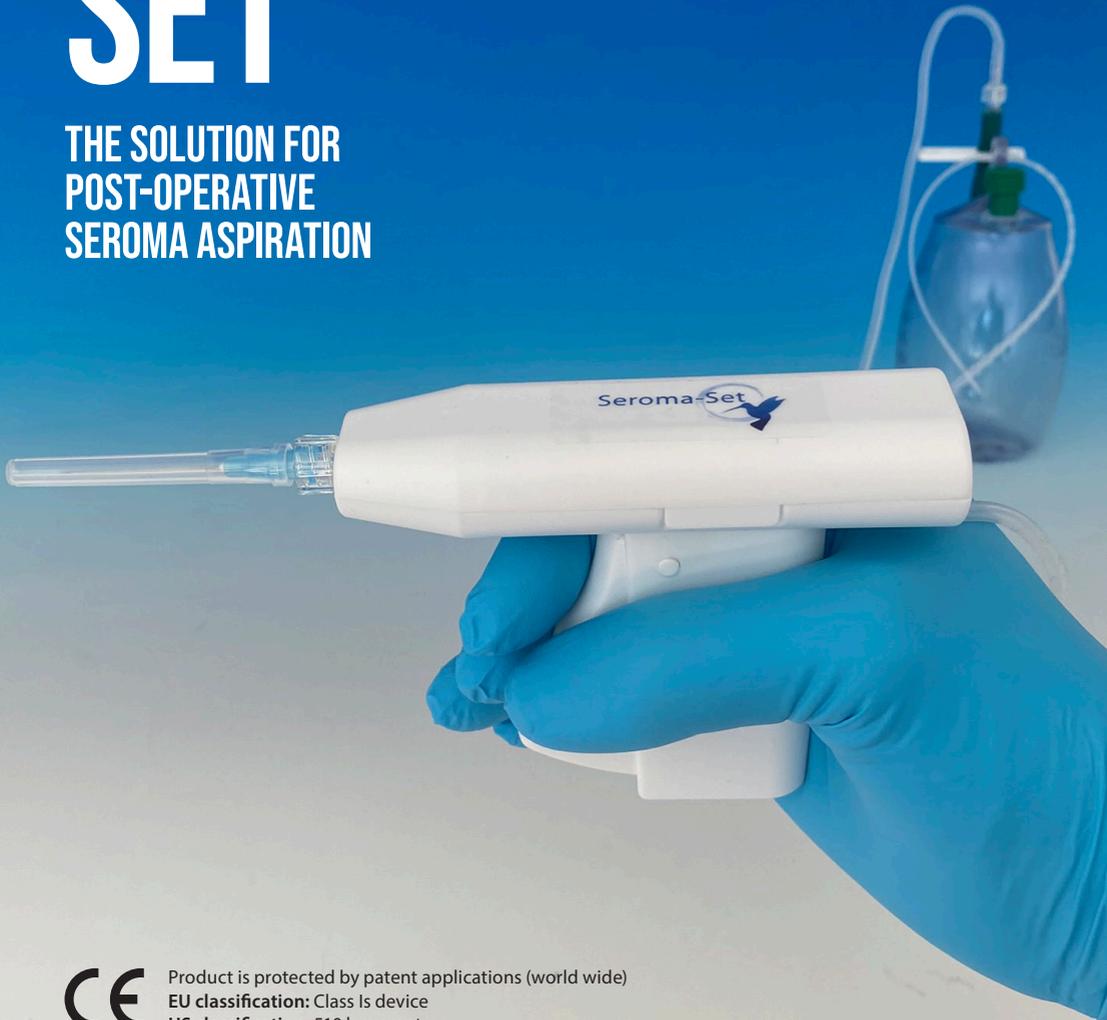

Product is protected by patent applications (world wide)
EU classification: Class Is device
US classification: 510 k exempt

SEROMA SET™

THE SOLUTION FOR POST-OPERATIVE SEROMA ASPIRATION

Patient friendly

Complete removal of seroma reduces number of aspirations
Lower risk of infection

User-friendly

Can be operated with one hand
Avoids the need for awkward syringe aspiration
No syringe required to create vacuum

Time-saving

Emptying seroma cavities is significantly faster than with a syringe/needle

PRODUCT CODES

Ref no 1030: SeromaSet™ 400 ml (5 pcs)

Ref no 1031: SeromaSet™ 600 ml (4 pcs)

Ref no 1060: SeromaSet™ 400 ml (25 pcs)

Ref no 1061: SeromaSet™ 600 ml (22 pcs)

Ref no 1020: Bottle 400 ml (44 pcs)

Ref no 1021: Bottle 600 ml (32 pcs)

Ref no 1050: Handset (100 pcs)

PRODUCT DESCRIPTION

The Seroma Set™ is a single use system for aspiration of seroma formation.

The system consists of:

- Handset which holds the needle and controls the vacuum.
- Vacuum container where the seroma fluid will be contained (we recommend PFM-Medical Redon Bottle or another high vacuum bottle - manufacturer instructions included in each Seroma Set™ box).

Safe and effective

Sterile single use – no contact with the aspirated fluid
Single attachment/detachment process may reduce risk of infection
Complete emptying of seroma from cavity in single action

Cost-effective

Fewer aspirations and reduced time to complete procedure

Note: When the Seroma Set™ is ready for use, please do not squeeze the trigger before the needle is inserted in to the seroma cavity. If activated outside the cavity, the vacuum is destroyed instantly and cannot remove the fluid.



Product designed and manufactured in Denmark

info@innoventamedica.com

SEROMA SET™

QUICK GUIDE | INSTRUCTIONS FOR USE



Unpack each item.



Check the vacuum indicator is compressed.



Connect the tube to the Luer-Lock on the vacuum container.



Attach a needle of choice.



Open the vacuum clamp and pinch the green tube to ensure free flow. The Seroma Set™ is ready for use.



FURTHER GUIDANCE

Scan the QR code for further instructions and guidance

PLEASE CONTACT YOUR LOCAL DISTRIBUTOR TO ORDER SEROMA SET™



Denmark, Germany, France, Slovakia, the Netherlands, Sweden and Austria
Contact: www.kebomed.dk, [de](http://www.kebomed.de), [fr](http://www.kebomed.fr), [sk](http://www.kebomed.sk), [nl](http://www.kebomed.nl), [se](http://www.kebomed.se), [at](http://www.kebomed.at)

Switzerland

Contact: **Knauer Medical**, www.knauer-medical.com

Ireland

Contact: **OLM**, www.olm.ie

United Kingdom

Contact: **Medical Devices in Healthcare Ltd**, info@mdih.co.uk

NEW TO SEROMA SET™?

Contact your local distributor to receive a free supply to evaluate in your clinic.

**THANK YOU FOR YOUR INTEREST IN SEROMA SET™,
PLEASE CONTACT US FOR FURTHER INFORMATION**

KEBOMED

Formervangen 5, DK-2600 Glostrup
(+45) 36398080, info@kebomed.dk
www.kebomed.dk